- User and maintenance instruction manual
- Manuel d'utilisation et entretien
- Manual operativo y de mantenimiento

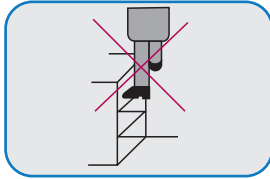
Keep these instructions close to the refrigerator.  
Garder ces instructions près du réfrigérateur.  
Guardar estas instrucciones cerca del refrigerador.

## GBR 50 AC - Blood bank Refrigerator

**MODEL NUMBER:** GBR 50 AC



Sure Chill® is a patented technology, licensed by The Sure Chill Company

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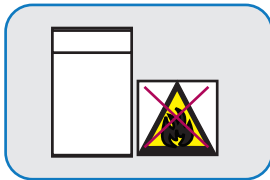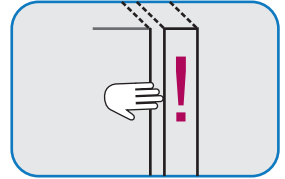
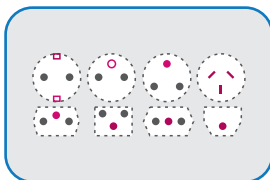
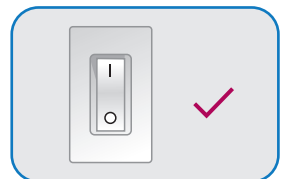
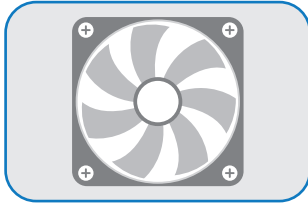
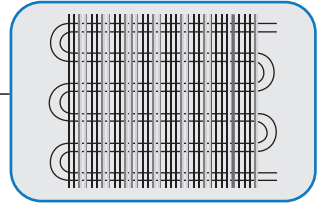
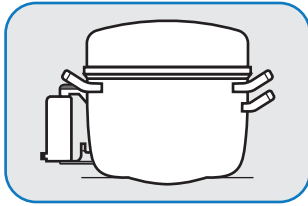
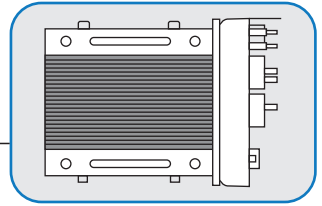
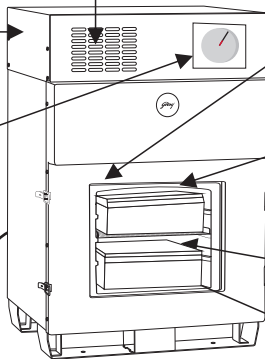
- Door seal
- Joint de porte
- Sello de la puerta

- Bins
- Bacs
- Cajones


- Compartment
- Compartiment
- Compartimiento

## OPERATING INSTRUCTIONS / PRÉCAUTIONS D'EMPLOI / INSTRUCCIONES DE FUNCIONAMIENTO

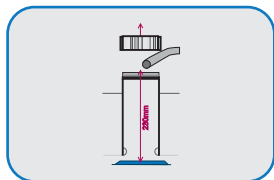
- Do not switch on appliance before filling with water. Remove ventilated lid and screw cap and fill with clean drinking water to 230 mm from top of pipe.
- Ne mettez pas l'appareil en marche sans l'avoir d'abord rempli d'eau. Retirez le bouchon à vis et remplissez d'eau potable propre jusqu'à 230 mm sous le bord du tube
- No encienda el aparato antes de llenarlo de agua. Quitar la tapa a rosca y llenar con agua potable limpia hasta los 230 mm desde la parte superior de la manguera

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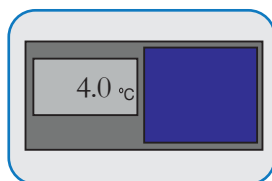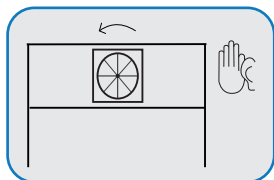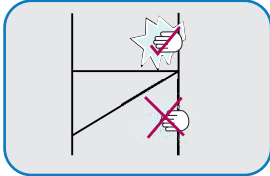
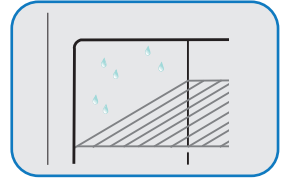
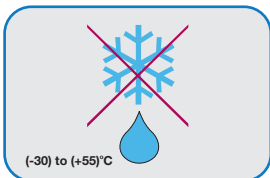
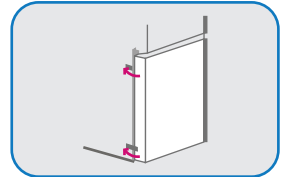
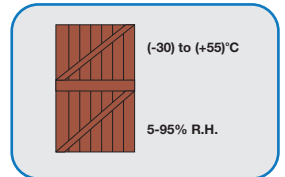
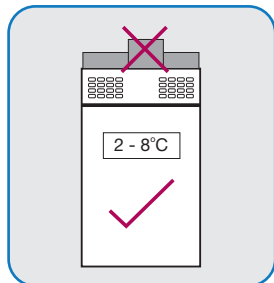
**Daily** - confirm compartment temperature is within acceptable range and check all vents are unobstructed

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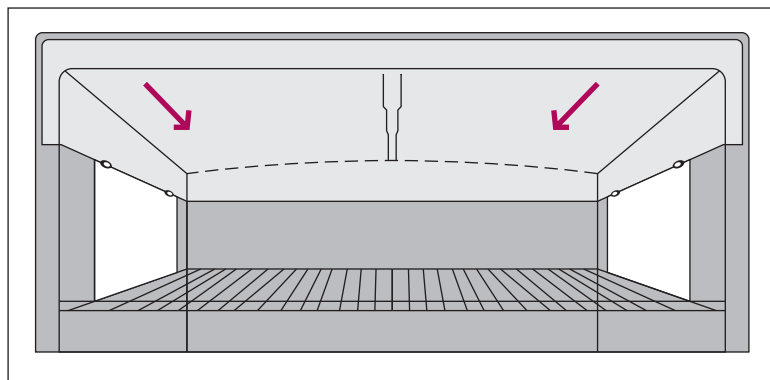
**Weekly** - Clean condensation baffle using soft cloth. Wipe clean and dry door seal and inside surfaces to reduce condensation buildup.

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The product has a built-in intelligent temperature monitoring, malfunction detection and early warning system. A Sensor sounds an audio alarm for 1 min every 15 minutes (power supply being available) if the cooling system has failed. In this case, call for technical assistance. Compartment temperature will be maintained between 2-6° C for some additional period post the alarm. In case of alarm, display temperature will have to be monitored frequently. Blood Bags stored in compartment will have to be transferred when safe storage temperatures limit is reached.

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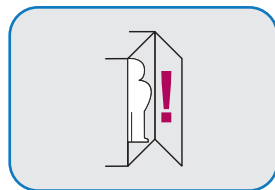
## CUSTOMER SUPPORT / SOUTIEN CLIENTELE / SERVICIO AL CLIENTE

Godrej & Boyce Manufacturing Company Limited,  
Pirojshanagar,  
Vikhroli West,  
Mumbai – 400 079  
<http://www.godrejappliances.com/medical>

Board: +91 22 6796 5656 / 5959  
Fax: +91 22 6796 1700 / 1800

E-mail:

Sales Support: [medicalref@godrej.com](mailto:medicalref@godrej.com) / [hello@surechill.com](mailto:hello@surechill.com)

Service Support: [smartcare@godrej.com](mailto:smartcare@godrej.com)