GVR
50 DC

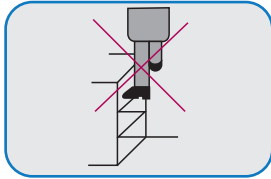
Version-V3

- User and maintenance instruction manual
- Manuel d'utilisation et entretie
- Manual de uso y mantenimiento

Keep these instructions close to the refrigerator.
Garder ces instructions près du réfrigérateur.
Guardar estas instrucciones cerca del refrigerador.

GVR 50 DC - Solar Direct Drive Vaccine Refrigerator
MODEL NUMBER: GVR 50 DC

SAFETY INSTRUCTIONS/ INSTRUCTIONS DE SÉCURITÉ/
INSTRUCCIONES DE SEGURIDAD

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

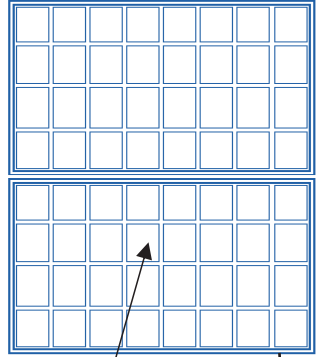
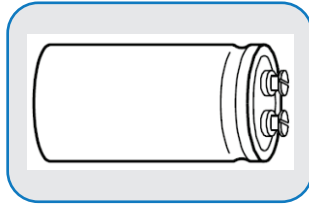
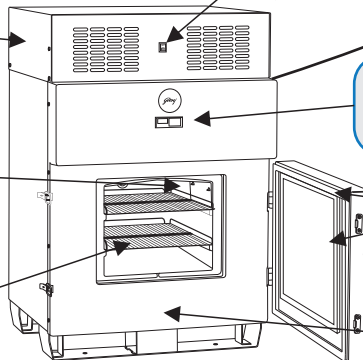
- Door seal
- Joint de porte
- Sello de la puerta

- Compartment
- Compartiment
- Compartimiento

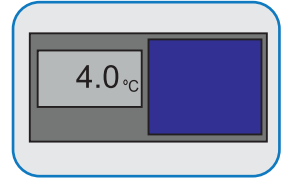- Compartment temperature is displayed on appliance in °C
- Thermometer is solar powered and requires light to operate correctly
- It is normal for appliance to be cooled to 4-6°C within 6-8 days after commissioning*

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
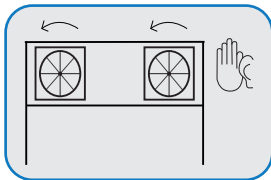
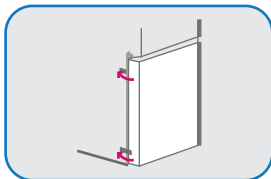
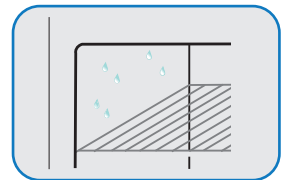
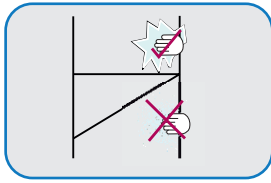
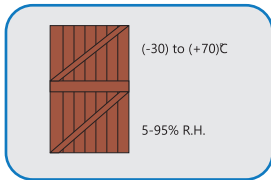
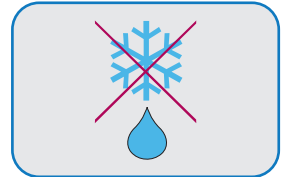
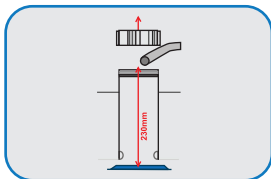
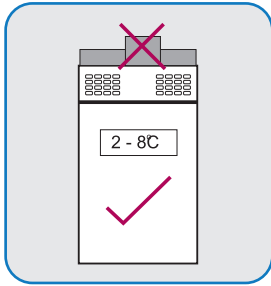
Weekly – Clean condensation baffle using soft cloth. Wipe clean and dry door seal and inside surfaces to reduce condensation buildup

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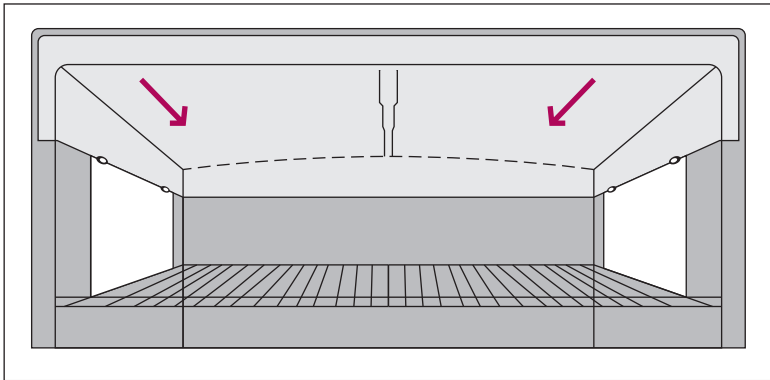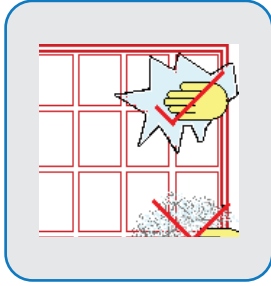Monthly – Check solar modules are unshaded and wipe with a clean cloth. This task should always be performed in the morning when the solar module is cold. Do not use any detergent on the solar modules

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Advice to customers for end of life disposal of our products:

- Each country will have its own laws regarding decommissioning of appliances, and customers are advised to adhere to relevant in-country procedures during the process of writing-off and physically disposing of equipment that is no longer effective. It is responsibility of the user to ensure that the product is disposed of properly. For more information, please visit: <http://www.surechill.com/disposal/>

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Godrej & Boyce Manufacturing Company Limited,
Pirojshanagar,
Vikhroli West,
Mumbai – 400 079
<http://www.godrejappliances.com/medical>

Board: +91 22 6796 5656 / 5959
Fax: +91 22 6796 1700 / 1800

E-mail:

Sales Support: medicalref@godrej.com / hello@surechill.com

Service Support: smartcare@godrej.com