GVR 55  
FF DC

Version-V0

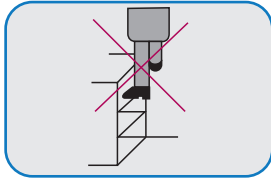
- User and maintenance instruction manual
- Manuel d'utilisation et entretie
- Manual de uso y mantenimiento

Keep these instructions close to the refrigerator.  
Garder ces instructions près du réfrigérateur.  
Guardar estas instrucciones cerca del refrigerador.

55101521SD01617

GVR 55 FF DC - Solar Direct Drive Vaccine Refrigerator  
MODEL NUMBER: GVR 55 FF DC

## SAFETY INSTRUCTIONS/ INSTRUCTIONS DE SÉCURITÉ/ INSTRUCCIONES DE SEGURIDAD

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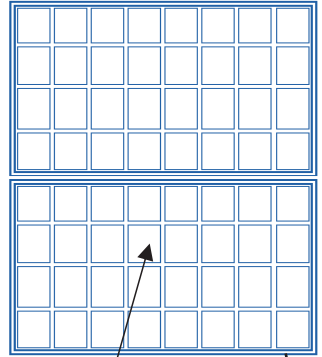
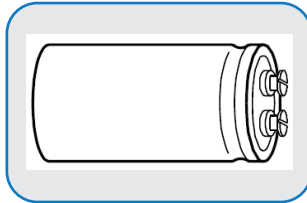
- Door seal
- Joint de porte
- Sello de puerta

- Compartment
- Compartiment
- Compartimiento

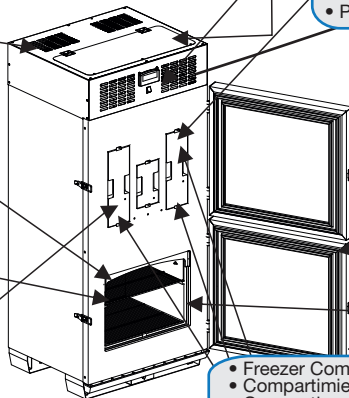
- Freezer Compartment
- Compartimiento del congelador
- Compartiment congélateur

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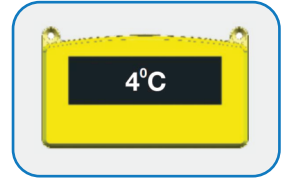
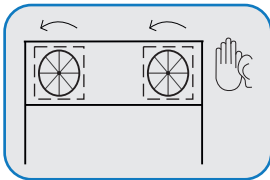
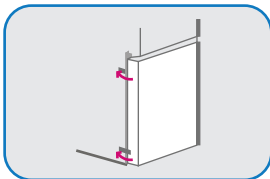
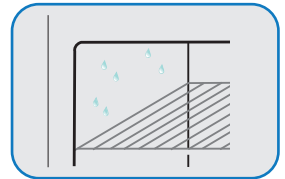
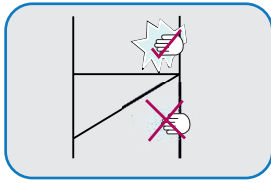
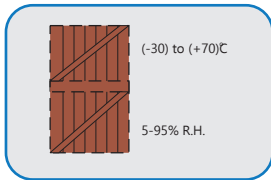
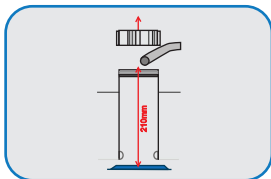
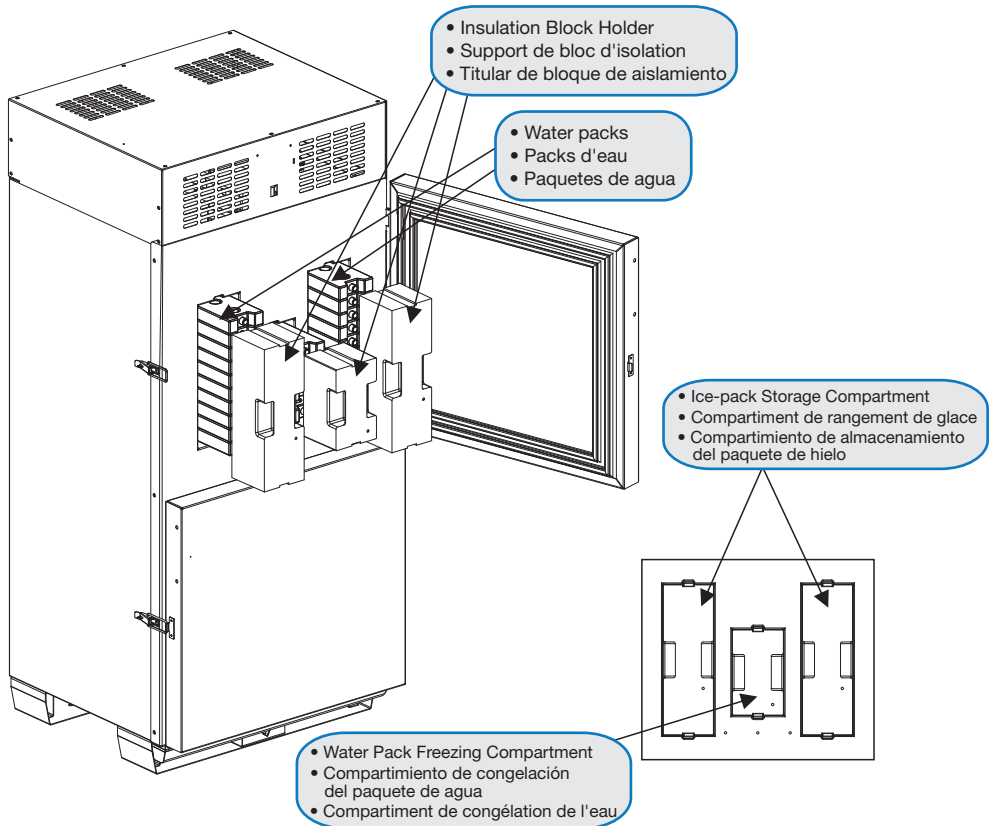
\*Good examples of clean water would be municipal water(mains water supply), Reverse Osmosis(RO) water, distilled water or clean, bottled drinking water. The water should not be dirty or contaminated. A mild dose of chlorine, consistent with drinking water quality standards, is helpful in disinfecting both the water and the container.

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The Freezer section on this product is designed in 3 sections- The Central Section designated as Freezer and the two side Sections designated as Storage. All the 3 sections, however, provide active freezing. Thus, the Ice packs can be frozen in the all 3 sections. In the Solar version of the Product, it could be observed at times after the night cycle, some of the packs could be marginally unfrozen or Partially frozen. This is essentially due to the night cycle.

It is recommended that the Semi frozen packs in the freezer section be transferred to the storage section even in this condition, where it will continue to freeze, as these sections also provide active freezing. The User, at the end of 6 days, will still get effectively 24 frozen icepacks for the outreach program carried out once a week.

The product delivers the core intent of availability of 24 fully frozen icepacks in 6 days. The system, however, may marginally not comply to the daily requirement completely as explained above.

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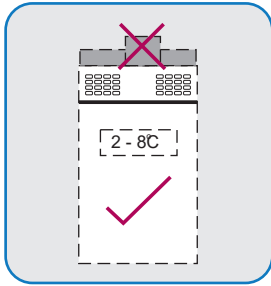
**Daily** - Confirm compartment temperature is within acceptable range and check all vents are unobstructed

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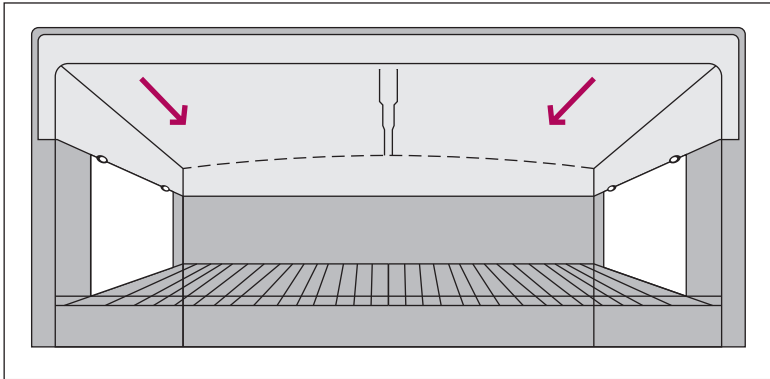
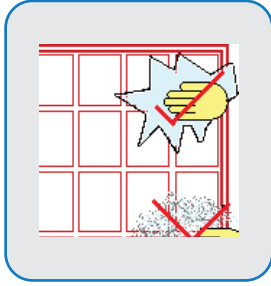
Weekly – Clean condensation baffle using soft cloth. Wipe clean and dry door seal and inside surfaces to reduce condensation buildup

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Monthly – Check solar modules are unshaded and wipe with a clean cloth. This task should always be performed in the morning when the solar module is cold. Do not use any detergent on the solar modules

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## PRODUCT DISPOSAL/ MISE AS REBUT DU PRODUIT/ DESCARTE DEL PRODUCTO

### Advice to customers for end of life disposal of our products:

- Each country will have its own laws regarding decommissioning of appliances, and customers are advised to adhere to relevant in-country procedures during the process of writing-off and physically disposing of equipment that is no longer effective. It is responsibility of the user to ensure that the product is disposed of properly. For more information, please visit: <http://www.surechill.com/disposal/>

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Godrej & Boyce Manufacturing Company Limited,  
Pirojshanagar,  
Vikhroli West,  
Mumbai – 400 079  
<http://www.godrejappliances.com/medical>

Board: +91 22 6796 5656/ 59 59  
Fax: +91 22 6796 1700 / 1800

E-mail:

Sales Support: [medicalref@godrej.com](mailto:medicalref@godrej.com) / [hello@surechill.com](mailto:hello@surechill.com)

Service Support: [smartcare@godrej.com](mailto:smartcare@godrej.com)