GVR
75 Lite

- User and maintenance instruction manual
- Manuel d'utilisation et entretien
- Manual operativo y de mantenimiento

Keep these instructions close to the refrigerator.
Garder ces instructions près du réfrigérateur.
Guardar estas instrucciones cerca del refrigerador.

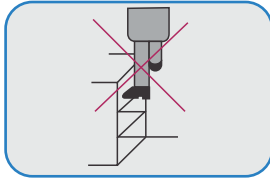
55101521SD00676

GVR 75 Lite

MODEL NUMBER: GVR 75 Lite



Sure Chill® is a patented technology, licensed by The Sure Chill Company

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

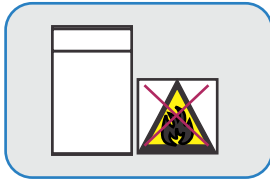
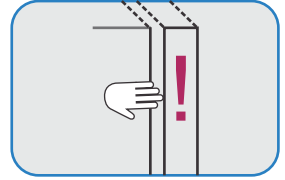
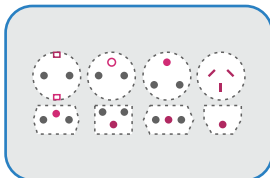
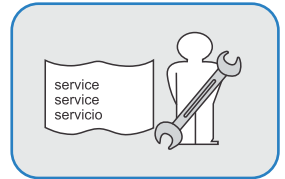
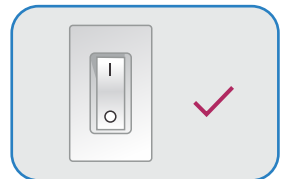
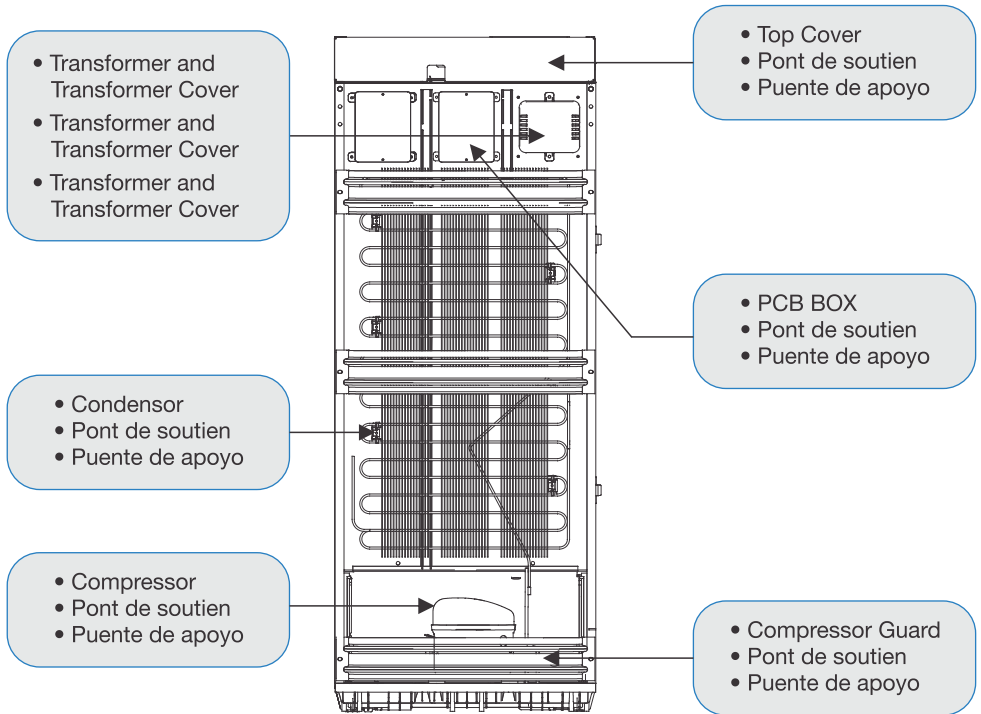
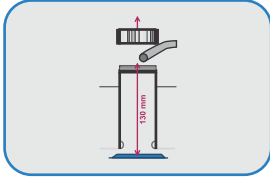
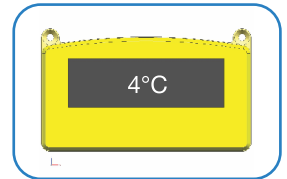
For a full description refer to the latest World Health Organisation publication on the subject: Guidelines for drinking-water quality, fourth edition
Editors: WHO
Publication date: 2011
Languages: English
ISBN: 978 92 4 154815 1
http://www.who.int/water_sanitation_health/publications/dwq_guidelines/en/

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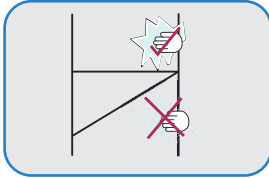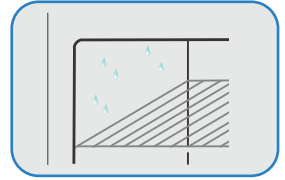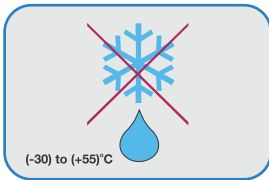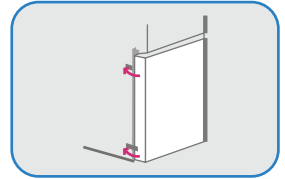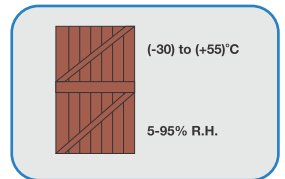
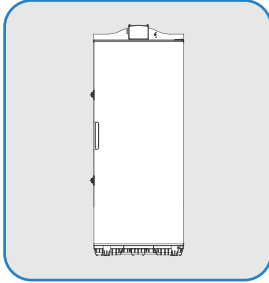
MAINTENANCE TASKS/ ENTRETIEN/ TAREAS DE MANTENIMIENTO

Daily - confirm compartment temperature is within acceptable range and check all vents are unobstructed

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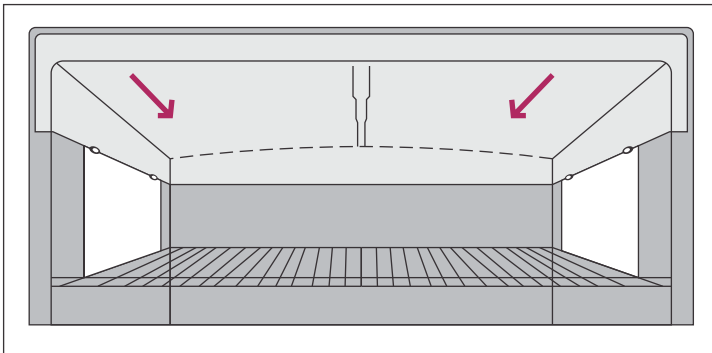
Weekly - Clean condensation baffle using soft cloth. Wipe clean and dry door seal and inside surfaces to reduce condensation buildup.

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The product has a built-in intelligent temperature monitoring, malfunction detection and early warning system. A Sensor sounds an audio alarm for 1 min every 15 mins (power supply being available) if the cooling system has failed. In this case, call for technical assistance. Compartment temperature will be maintained between 2-8°C for some additional period post the alarm. In case of alarm, display temperature will have to be monitored frequently. Vaccines stored in compartment will have to be transferred when safe storage temperatures limit is reached.

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PRODUCT DISPOSAL/ MISE AS REBUT DU PRODUIT/ DESCARTE DEL PRODUCTO

Advice to customers for end of life disposal of our products:

- Each country will have its own laws regarding decommissioning of appliances, and customers are advised to adhere to relevant in-country procedures during the process of writing-off and physically disposing of equipment that is no longer effective.

It is responsibility of the user to ensure that the product is disposed of properly.

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CUSTOMER SUPPORT/ SOUTIEN CLIENTELE/ SERVICIO AL CLIENTE

Godrej & Boyce Manufacturing Company Limited,
Pirojshanagar,
Vikhroli West,
Mumbai – 400 079
<http://www.godrejappliances.com/medical>

Board: +91 22 6796 5656 / 5959
Fax: +91 22 6796 1700 / 1800

E-mail:
Sales Support: medicalref@godrej.com / hello@surechill.com
Service Support: smartcare@godrej.com